**HELICOBACTER PYLORI
IgA IgG IgM Auro•Dex®
Rapid Test**

**HELICOBACTER PYLORI Dexall® Auro-Dex®
LATERAL FLOW RAPID TEST**

Unique One-Step Lateral Flow Test for the Rapid Detection of Helicobacter Pylori-Specific IgA, IgG, and IgM Antibodies in human serum

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HELICOBACTER PYLORI IgA IgG IgM Rapid Test

UNIQUE ONE-STEP TEST FOR THE RAPID DETECTION OF HELICOBACTER PYLORI-SPECIFIC IGA, IGG, AND IGM ANTIBODIES.

Helicobacter pylori is a gram-negative, spiral-shaped bacterium with unusually strong urease production. It colonises the luminal side of the epithelial cells of the stomach mucosa. *Helicobacter pylori* is the only human pathogenic type of the genus *Helicobacter* and is distributed worldwide. Humans and animals secrete the bacterium in their faeces. *Helicobacter pylori* is capable of surviving in water. With a prevalence of around 50 % worldwide, *Helicobacter pylori* infection is one of the most common chronic bacterial infections. The infection level is much higher in developing countries than in industrial countries. After contact with *Helicobacter pylori*, patients may exhibit antibodies of classes IgA, IgG and IgM against *Helicobacter pylori* in the serum. **IgA antibodies** usually form after a few weeks and remain detectable over a long time period. Positive IgA results correlate well with gastritis activity. However, these antibodies are formed locally and are not always detectable in the serum. **IgG antibodies** are frequently first detected after the IgM titer has declined and they can persist for years. An increase in the IgG titer indicates an advanced *Helicobacter pylori* infection. Elevated IgG antibody titers are considered a marker for chronic infection. **IgM antibodies** are formed a few days after contact with *Helicobacter pylori*. After a few weeks specific IgM is no longer detectable.

H. PYLORI Auro•Dex[®]

The **Auro•Dex[®] H.Pylori MultiTest** is a lateral flow immunochromatographic screening test. In the assay the test serum is allowed to react with H.Pylori antigen conjugated to gold particles. The mixture then flows laterally on the membrane to a test region (G, A, M) where immobilized antibodies (anti-IgG, anti-IgA, anti-IgM) capture the serum antibody-gold conjugated H.Pylori complex. A red colored line formed in the test region indicates the presence and class of H.Pylori antibodies in the test specimen. Absence of the colored line in the test region indicates a negative result. A red colored line in the control region (C) will always appear regardless of the presence or absence of H.Pylori specific antibodies in tested serum.

PERFORMANCE:

H.PYLORI Auro•Dex[®] lateral flow

Sensitivity: 95.9%

Relative Specificity: 97.4%

EUROPE Ordering Information:
HELICOBACTER PYLORI MultiTest
Product Code: D9441
Packaging: 15 test

Meridian Healthcare[®]

68 - Tremestieri Etneo

Catania, IT 95030

Phone: 095 -725 68 69 (0039)